

Child Positioner for GE CT Systems

Instructions For Use



Important Information

EN – E8003S CHILD POSITIONER FOR GE CT SYSTEMS

fr - E8003S Positionneur pour enfant pour systèmes CT de GE
de - E8003S Kinderpositionierer für GE CT-Systeme
it - E8003S Posizionatore da bambino per sistemi GE CT
es - E8003S Posicionador infantil para sistemas de tomografía computada GE
el - E8003S Τοποθετητής παιδιού για συστήματα αξονικού τομογράφου GE CT
ga - E8003S Socraitheoir Páiste do Chórais GE CT
pt-BR - E8003S posicionamento para crianças para utilização nos sistemas de TC da GE
ZH-CN - E8003S GE CT 系統的兒童體位固定裝置
ja - E8003S GE CTシステム用チャイルドポジショナ
bg - E8003S Детски позиционер за GE CT системи
zh - E8003S GE CT 系统的儿童体位固定装置
hr - E8003S Dječja kolijevka za uporabu sa GE CT sustavom
cs - E8003S Pediatrická Kolébka Kit pro systémy GE CT
da - E8003S Positioneringsenhed til børn til GE CT Systemer
nl - E8003S Child Positioner voor GE CT Systemen
et - E8003S Lapse positsioneerija GE KT-süsteemidele
fi - E8003S Lasten asetin GE TT -järjestelmiä varten
mt - E8003S Pożizzjonatur tat-Tfal għal Sistemi GE CT
hu - E8003S Gyermekek teströgzítő szett GE CT rendszerekhez
ko - E8003S GE CT 시스템 용 차일드 포지셔너
lv - E8003S Bērnu fiksators GE CT sistēmām
lt - E8003S Vaiko padėties nustatymo įtaisas, skirtas „GE“ KT sistemoms
no - E8003S Barneposisjoner for GE CT-systemer
pl - E8003S Pozycjoner pediatryczny do systemów TK firmy GE
pt - E8003S posicionamento para crianças para utilização nos sistemas de TC da GE
ro - E8003S Pozitie pentru copii pentru sistemele GE CT
ru - E8003S Детский позиционер для систем GE CT
sr - E8003S Дечји позиционер за GE ЦТ системе
id - E8003S Positioner Anak untuk Sistem GE CT
sk - E8003S Detský polohovateľ pre CT systémy GE
sl - E8003S Komplet za nsramestitev otroka v GE CT
sv - E8003S Barnfixering för GE CT-system
tr - E8003S GE CT Sistemleri için Çocuk Konumlandırıcı
uk - E8003S Дитячий позиционер для систем GE CT
kk - E8003S GE CT жүйелерінде пайдаланылатын баланы бекіткіш жиынтықтамасы
vi - E8003S Thiết bị Cố định Trẻ em cho Hệ thống GE CT



Manufactured By:
Domico Med-Device, LLC
14241 North Fenton Rd.
Fenton, MI 48430 USA
810-750-5300
DomicoMed.com



DomicoMed.com/eifu



**Not Made With
Natural Rubber
Latex**




Advena Ltd., Tower Business Centre, 2nd Flr.,
Tower Street, Swatar, BKR 4013 Malta



WARNING:

This product can expose you to chemicals which is known to the State of California to cause reproductive harm. For more information, go to www.P65Warnings.ca.gov. 1-methyl-2-pyrroldione (NMP) CAS: 872-50-4, EC: 212-828-1; Trixylyl phosphate (TXP) CAS: 25155-23-1, EC: 246-677-8

Components

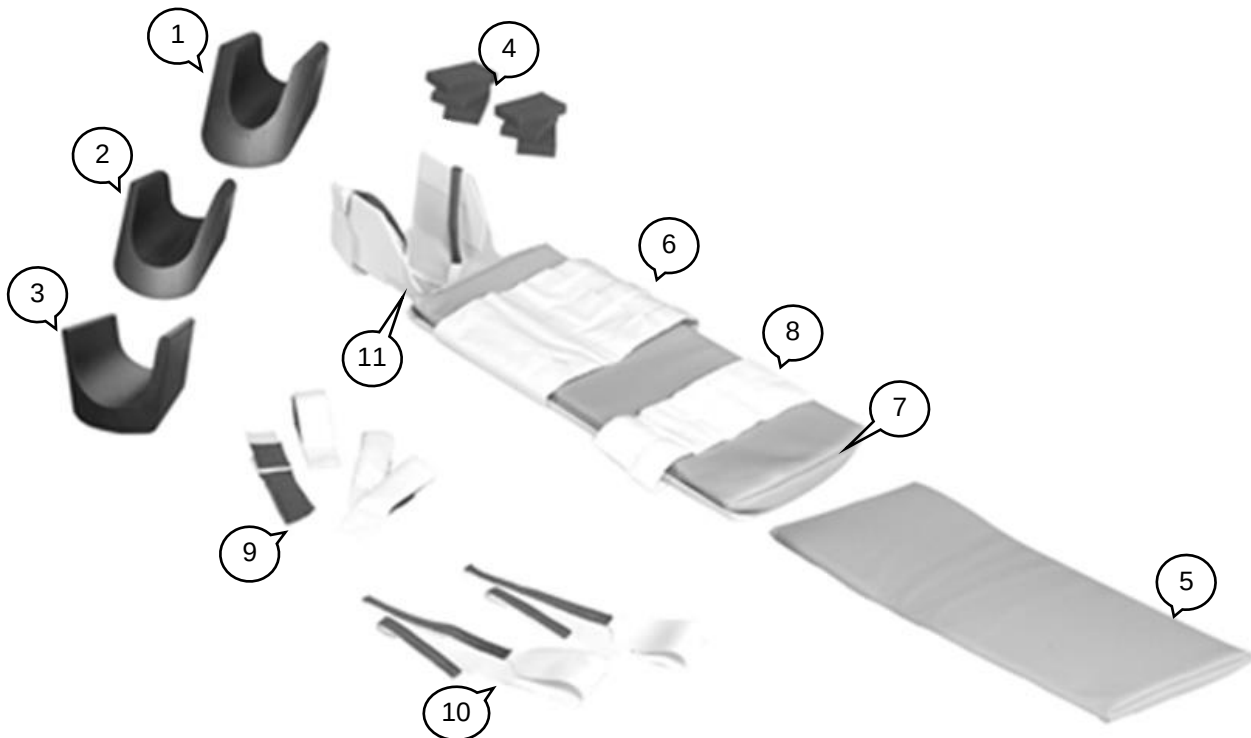

Item	Part Number	Description
1	28024	E8003SA CT Child Head Cushion - Small
2	28025	E8003SB CT Child Head Cushion - Medium
3	28026	E8003SC CT Child Head Cushion - Large
4	20603	E8003DG Foam Block
5	28060	E8005BD Pad, Extension
6	23802	E8005AG Child Positioner Body Strap - Wide
7	28057	E8005AJ Child Positioner Pad
8	28058	E8005AH Child Positioner Body Strap - Narrow
9	26940	E8005BA Child Positioner Wrist Straps
10	23531	E8002BB Child Positioner Head Straps
11	21540	E8005AK Child Positioner Cradle

Note: Any serious incident related to the use of the device should be reported to the manufacturer and the health authority/competent authority where the device is installed.




Warning! Patient board must be stable prior to loading patient to prevent patient fall.



Warning! Prior to starting procedure, patient must be appropriately secured into positioner, and positioner must be secured to table with straps to prevent patient fall.

Setup


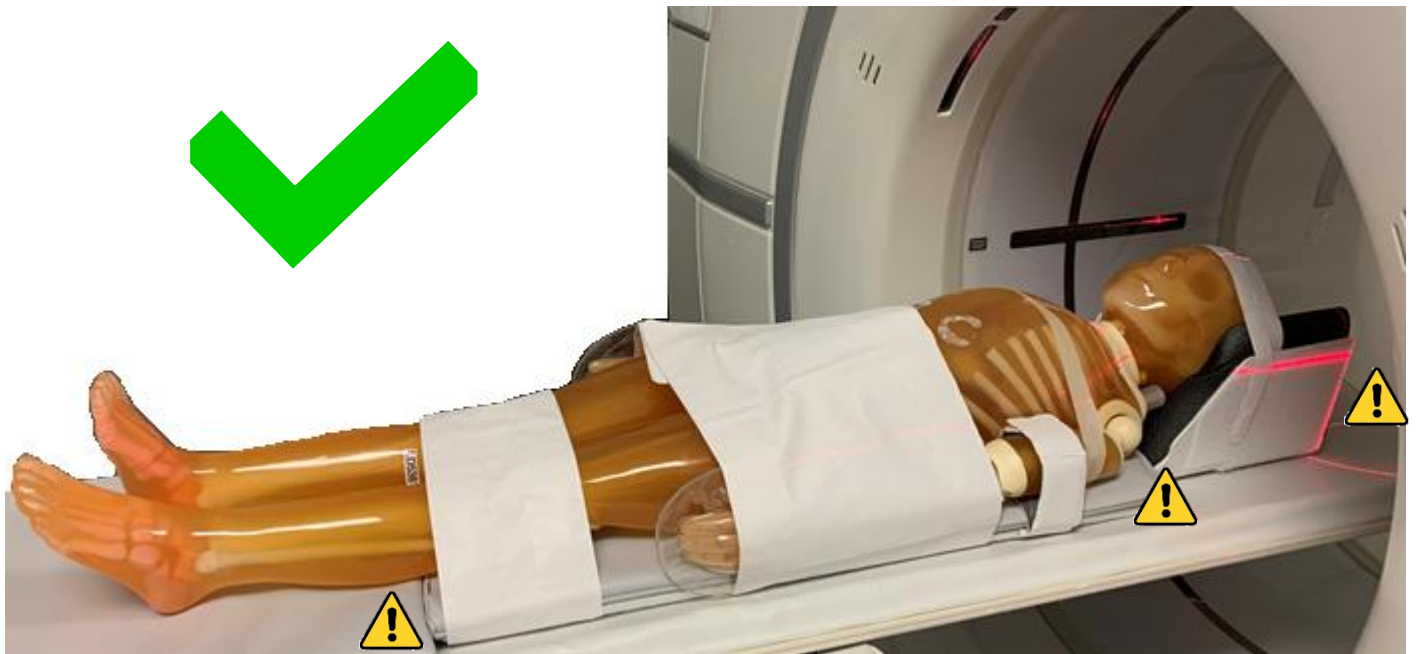




 **Warning!** Do not position anatomy of interest over junctions.






The positioner should be about ~12in or ~30.5cm from the end of the table so that there is no interference from the table extender latch.

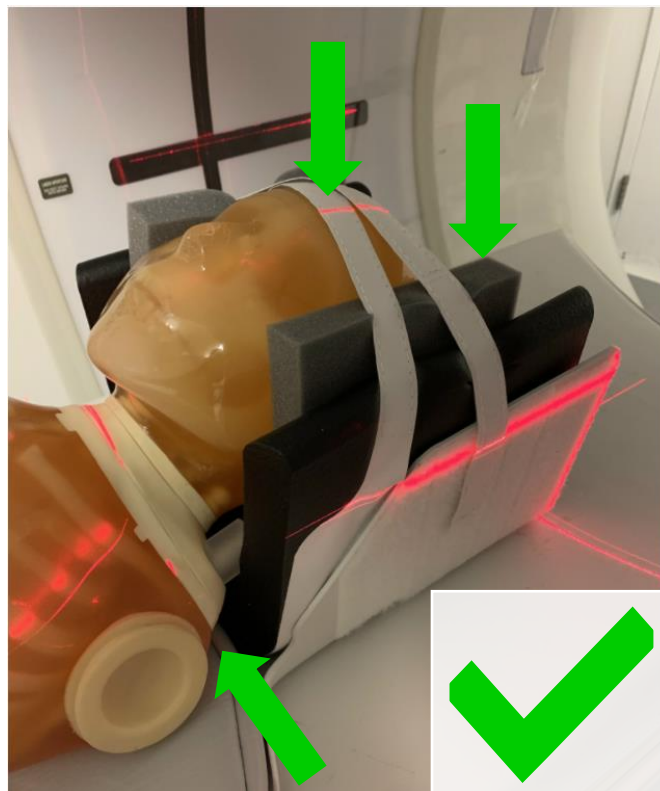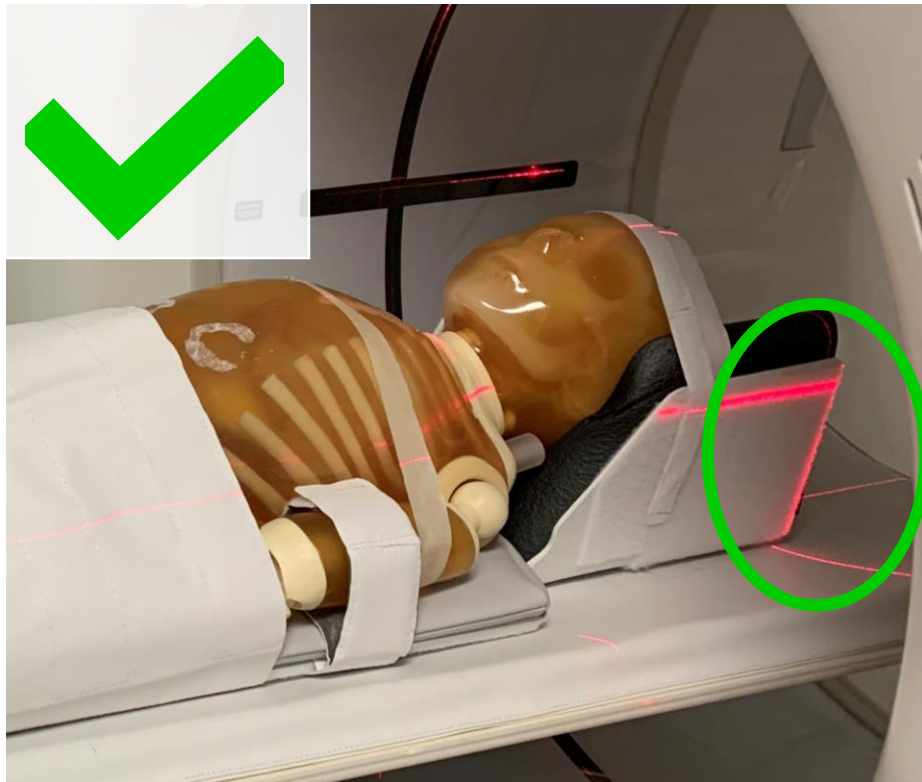

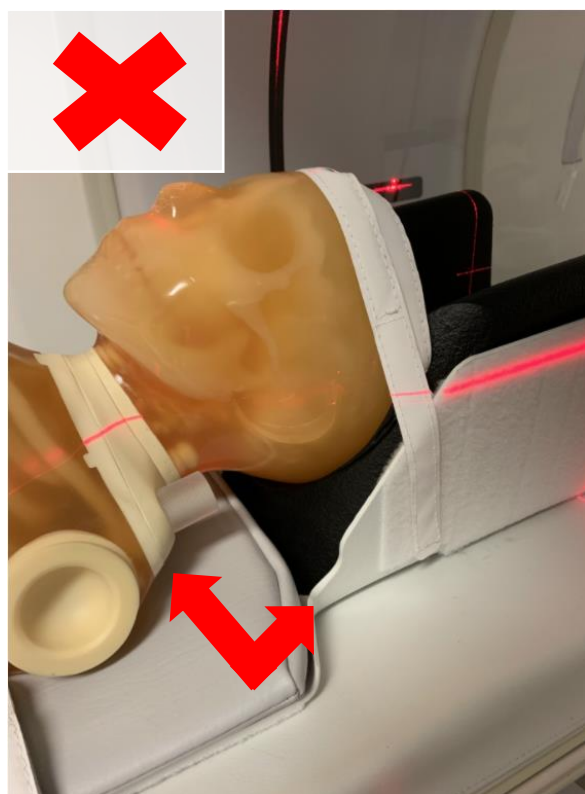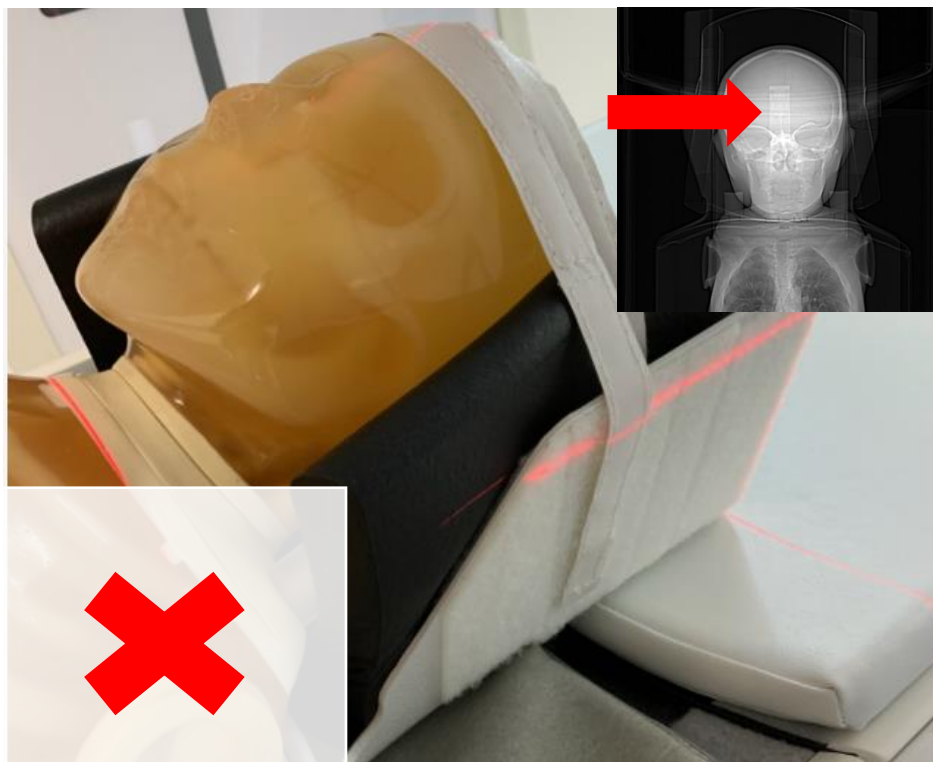

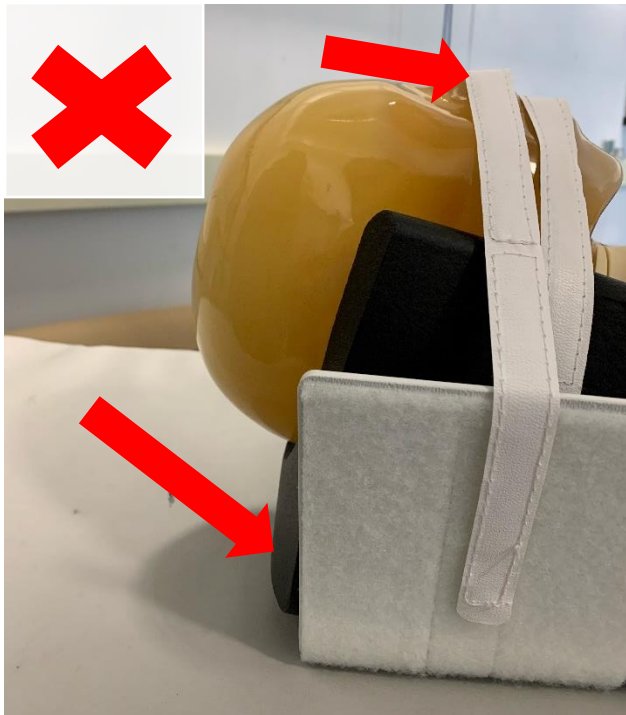

Cleaning and Disinfection

Clean by removing soil and disinfect per manufacturer's instruction/according to local facility environmental cleaning practices using:

- Common household bleach - diluted 10:1 (5-minute minimum disinfection time)
- Sani-Cloth® Bleach Wipes by PDI
- Bleach Germicidal Wipes by Clorox Healthcare®
- Bleach Germicidal Cleaner Spray by Clorox Healthcare®
- Hydrogen Peroxide Cleaner Disinfectant Wipes by Clorox Healthcare®
- Clorox® Multi Surface Cleaner With Bleach
- Micro-Kill Bleach Germicidal Bleach Solution by Medline
- Micro-Kill Bleach Germicidal Bleach Wipes by Medline



Warning! ALWAYS rinse products with clear water to remove any residue from cleaning/disinfecting solution. Failure to do so could result in patient injury.