

Child Positioner for use with Systems: Premier, Voyager, Explorer Instructions For Use



Important Information

EN - E8823MR Child Positioner

For Use With Systems: Premier, Voyager, Explorer



fr - E8823MR Positionneur pour enfant pour systèmes : Premier, Voyager, Explorer
de - E8823MR Kinderpositionierer zur Verwendung mit Systemen: Premier, Voyager, Explorer
it - E8823MR Posizionatore da bambino per l'uso con i sistemi: Premier, Voyager, Explorer
es - E8823MR Posicionador infantil para usar con los sistemas: Premier, Voyager, Explorer
el - E8823MR Τοποθετητής παιδιού για χρήση με τα συστήματα: Premier, Voyager, Explorer
ga - E8823MR Socraitheoir Páiste le húsáid in éineacht le Córais: Premier, Voyager, Explorer
pt-BR - E8823MR Kit de posicionamento para crianças para utilização com os sistemas: Premier, Voyager, Explorer
ZH - E8823MR 兒童體位固定裝置 - 適用系統 : Premier、Voyager、Explorer
ja - E8823MR Premier、Voyager、Explorer システム対応 チャイルドポジショナー
bg - E 8823MR Детски позиционер за използване със системи: Premier, Voyager, Explorer
zh - E8823MR 兒童體位固定裝置 - 適用系統 : Premier、Voyager、Explorer
hr - E8823MR Dječja kolijevka za uporabu sa sustavima: Premier, Voyager, Explorer
cs - E8823MR Dětská kolébka Kit Pro použití se systémy: Premier, Voyager, Explorer
da - E8823MR Positioneringsenhed til børn til systemerne: Premier, Voyager, Explorer
nl - Child Positioner voor gebruik met: Premier, Voyager, Explorer systemen
et - Lapse positsioneerija süsteemidele Premier, Voyager, Explorer
fi - E8823MR Lasten asetin käytettäväksi seuraavien järjestelmien kanssa: Premier, Voyager, Explorer
mt - E8823MR Pożizzjonatur tat-Tfal biex jintuża mas-Sistemi: Premier, Voyager, Explorer
hu - E8823MR Gyermek test rögzítő, amely az alábbi rendszerekkel használható: Premier, Voyager, Explorer
ko - E8823MR 자식 포지셔너시스템과 함께 사용하기 위해: Premier, Voyager, Explorer
lv - E8823MR Bēmu fiksators lietošanai ar sistēmām: Premier, Voyager, Explorer
lt - E8823MR Vaiko padėties nustatymo įtaisas, skirtas naudoti su sistemomis „Premier“, „Voyager“, „Explorer“
no - E8823MR Underordnet stillingsstiller For bruk med systemer: Premier, Voyager, Explorer
pl - E8823MR Pozycjoner pediacyjny do użytku z systemami: Premier, Voyager, Explorer
pt - E8823MR Kit de posicionamento para crianças para utilização com os sistemas: Premier, Voyager, Explorer
ro - E8823MR Kit de leagăn pediatric pentru utilizare cu sisteme: Premier, Voyager, Explorer
ru - E8823MR Детский позиционер Для использования с системами: Premier, Voyager, Explorer
sr - E8823MR Дечији позиционер За употребу са системима: Premier, Voyager, Explorer
id - E8823MR Posisi anak Untuk Digunakan dengan Sistem: Premier, Voyager, Explorer
sk - E8823MR Detský polohovateľ na použitie so systémami: Premiér, Voyager, Explorer
sl - E8823MR Komplet za namestitev otroka v sistemih: Premier, Voyager, Explorer
sv - E8823MR Barnfixering för användning med systemen: Premier, Voyager, Explorer
tr - E8823MR Alt konumlayıcı Sistemlerle Kullanım İçin: Premier, Voyager, Explorer
uk - E8823MR Дочірній позиціонер Для використання з системами: Premier, Voyager, Explorer
kk - E8823MR Premier, Voyager, Explorer жүйелерінде пайдаланылатын баланы бекіткішті пайдалану жөніндегі нұсқаулық
vi - E8823MR Thiết bị Cố định Trẻ em để sử dụng với các Hệ thống: Premier, Voyager, Explorer



Manufactured By:
Domico Med-Device, LLC
14241 North Fenton Rd.
Fenton, MI 48430 USA
810-750-5300
DomicoMed.com



DomicoMed.com/eifu



**Not Made With
Natural Rubber
Latex**




Advena Ltd., Tower Business Centre, 2nd Flr.,
Tower Street, Swatar, BKR 4013 Malta



WARNING:

This product can expose you to chemicals which is known to the State of California to cause reproductive harm. For more information, go to www.P65Warnings.ca.gov. 1-methyl-2-pyrroldione (NMP) CAS: 872-50-4, EC: 212-828-1; Trixylyl phosphate (TXP) CAS: 25155-23-1, EC: 246-677-8

Components

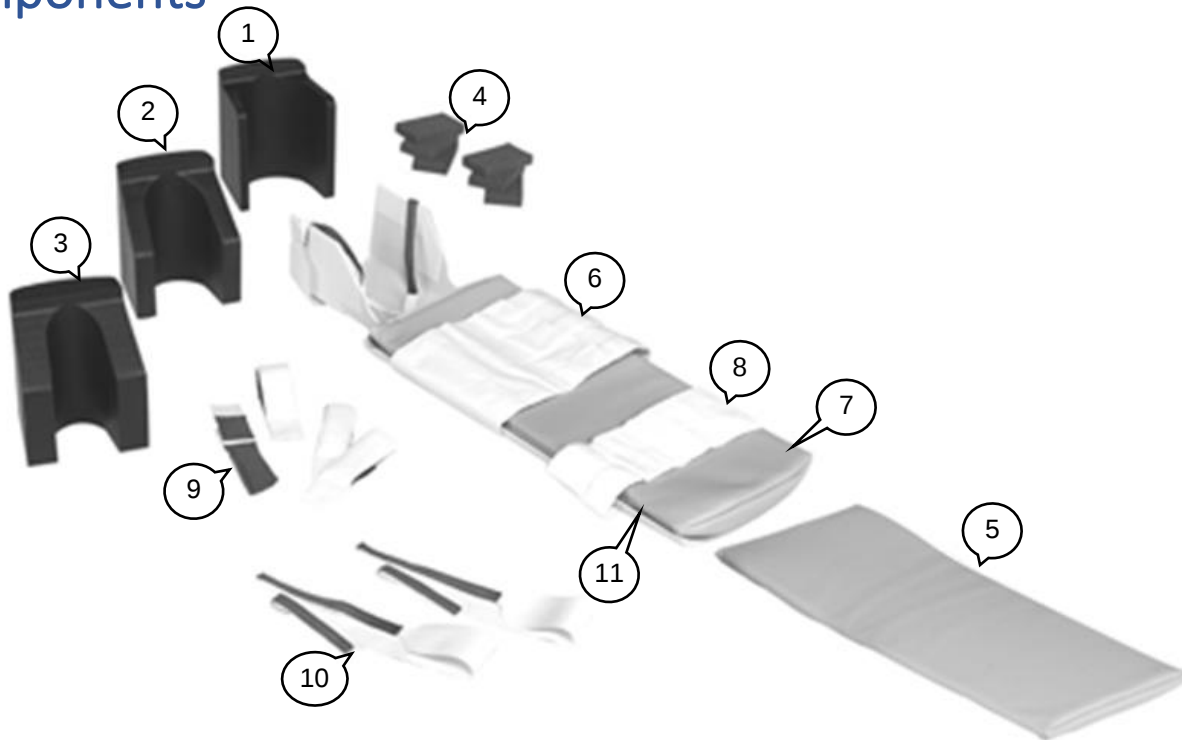

Item	Part Number	Description
1	21349	E8005AN MR/NUC Child Head Insert, Large
2	21350	E8005AM MR/NUC Child Head Insert, Medium
3	21351	E8005AL MR/NUC Child Head Insert, Small
4	20603	E8003DG Foam Block
5	28060	E8005BD Pad, Extension
6	23802	E8005AG Child Positioner Body Strap, Wide
7	28057	E8005AJ Child Positioner Pad
8	28058	E8005AH Child Positioner Body Strap - Narrow
9	26940	E8005BA Child Positioner Wrist Straps
10	23531	E8002BB Child Positioner Head Straps
11	21540	E8005AK Child Positioner Cradle

Note: Any serious incident related to the use of the device should be reported to the manufacturer and the health authority/competent authority where the device is installed.




Warning! Patient board must be stable prior to loading patient to prevent patient fall.



Warning! Prior to starting procedure, patient must be appropriately secured into positioner, and positioner must be secured to table with straps to prevent patient fall.

Setup

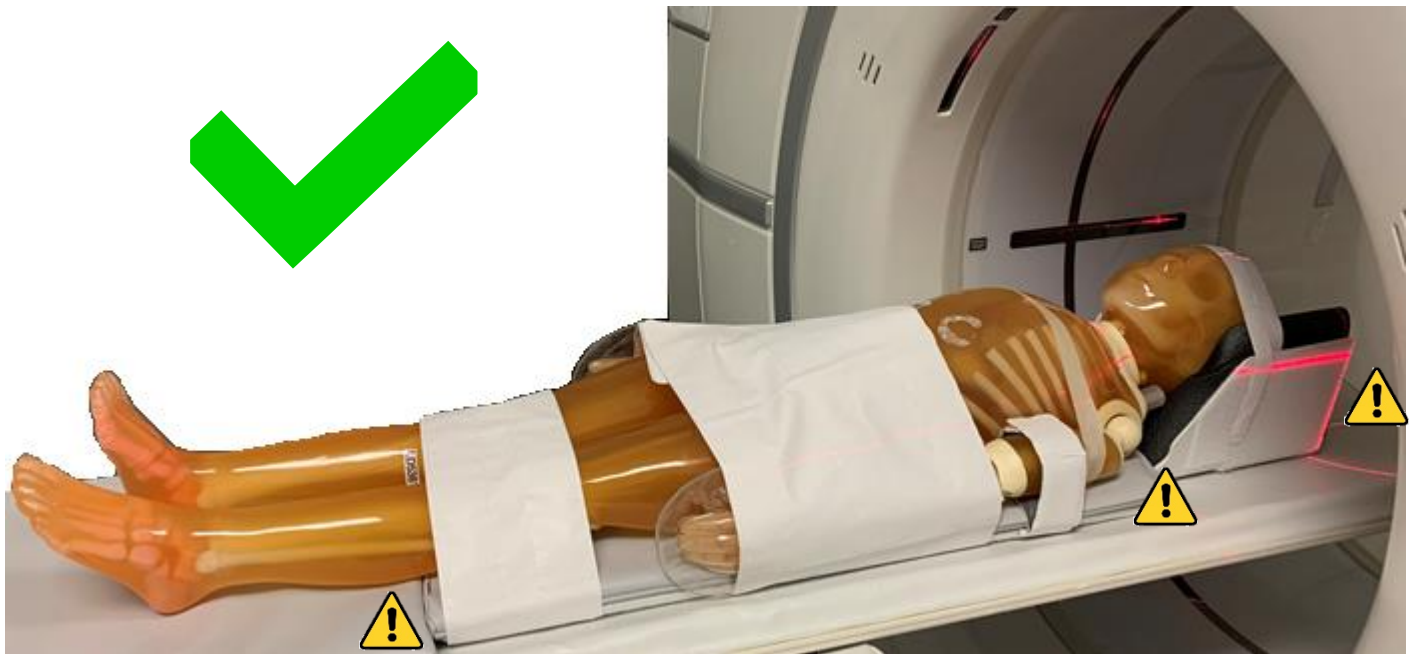




 **Warning!** Do not position anatomy of interest over junctions.





The positioner should be about ~12in or ~30.5cm from the end of the table so that there is no interference from the table extender latch.

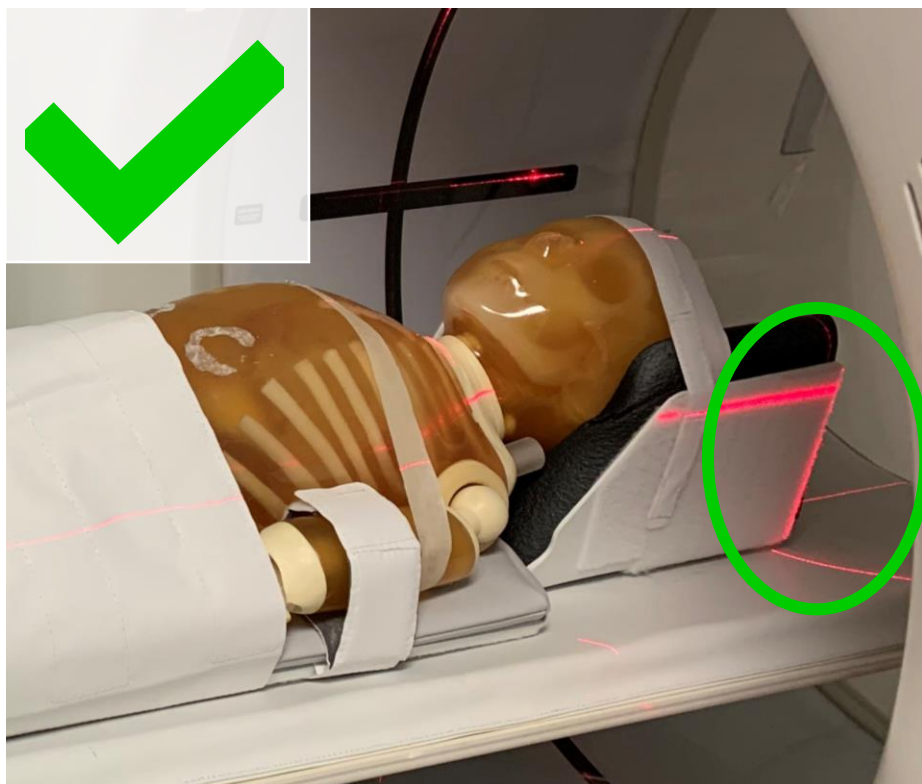

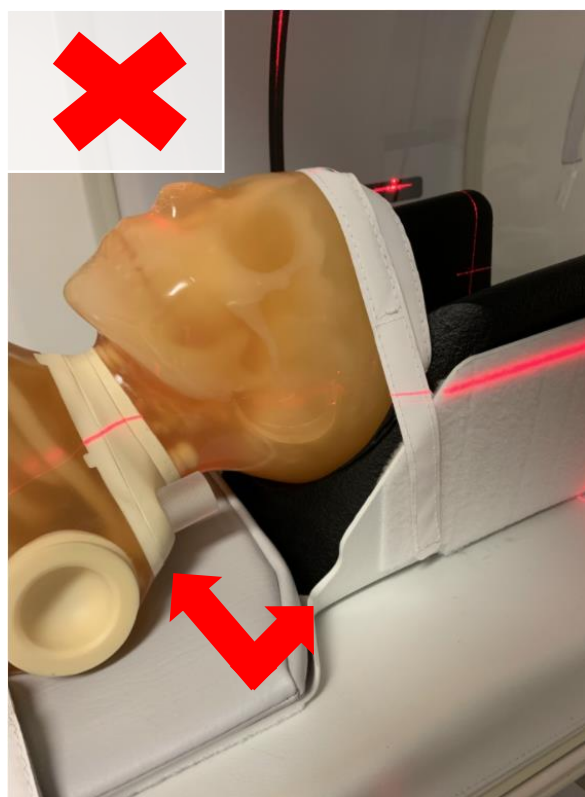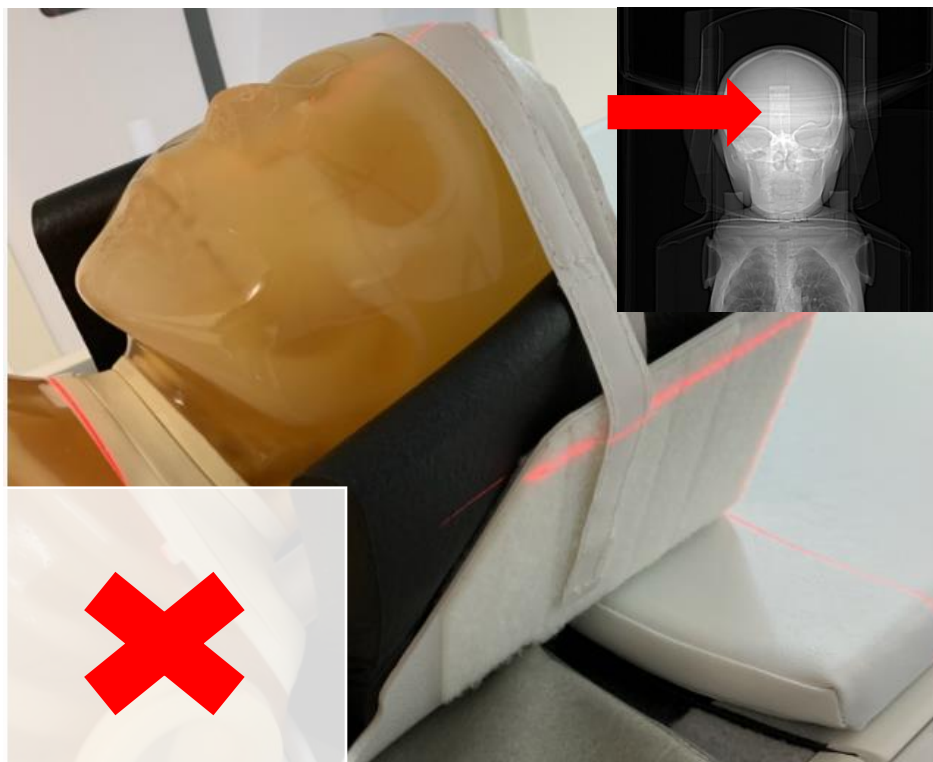



Cleaning and Disinfection

Clean by removing soil and disinfect per manufacturer's instruction/according to local facility environmental cleaning practices using:

- Common household bleach - diluted 10:1 (5-minute minimum disinfection time)
- Sani-Cloth® Bleach Wipes by PDI
- Bleach Germicidal Wipes by Clorox Healthcare®
- Bleach Germicidal Cleaner Spray by Clorox Healthcare®
- Hydrogen Peroxide Cleaner Disinfectant Wipes by Clorox Healthcare®
- Clorox® Multi Surface Cleaner With Bleach
- Micro-Kill Bleach Germicidal Bleach Solution by Medline
- Micro-Kill Bleach Germicidal Bleach Wipes by Medline



Warning! ALWAYS rinse products with clear water to remove any residue from cleaning/disinfecting solution. Failure to do so could result in patient injury.